

amigo

Aluminum Walker

code : M200



 mobilita

www.mobilita.in
TollFree : 1800 121 8485



www.mobilita.in
TollFree : 1800 121 8485

 mobilita

amigo Aluminum walker

USER MANUAL



Read the instruction manual as well as the mounting instructions before mounting and using it. Observe the safety warnings and make sure that only persons familiar with them use the walker. Save the instruction manual for future use and attach it if you transfer the product.

INDEX OF CONTENTS

- 01. Packaging content / Description of parts
- 02. Technical information
- 03. Symbols used
- 04. Indications of use
- 05. Security instructions
- 06. Mounting and adjustment
- 07. Mounting
 - Adjusting the handle height
 - Shopping bag
 - Seat
 - Care, maintenance and storage
- 08. Accessories and spare parts

PACKAGE CONTENT



- 1. **Handle**
- 2. **Backrest**
- 3. **Adjuster for height adjustable push handle**
- 4. **Tube frame with wheels**
- 5. **Front bar**
- 6. **Shopping bag**
- 7. **Seat**
- 8. **Base frame**

Symbols Used



Observe the instruction manual.



Maximum load on the seat



Manufacturing date



Observation

TECHNICAL INFORMATION

Maximum user weight		135kg.
Measures in unfolded position	Width	60cm / 23.62"
	Depth	57cm / 22.44"
Overall height	From	76cm / 30.00"
	To	95cm / 37.40"
Seat measurement	Width	37cm / 14.57"
	Depth	23cm / 9.25"
Seat height	From	46cm / 18.30"
	To	52cm / 20.67"
Material	Reinforced aluminum frame Nylon shopping bag PU seat	
Weight	Approximately 6 kg.	

INDICATIONS OF USE

The use of this mobility product is recommended in the following cases:

- Indicated for people with reduced mobility since it increases the freedom of movement with respect to the basic necessities of moving at home; Indoor use
- The need to leave the house to take the air during a walk or to make daily managements near home; Outdoor use.
- The walker is indicated for use as a seat for short periods



SECURITY INSTRUCTIONS

- Do not allow children to play with packaging without supervision. Risk of suffocation for children!
- Keep the product away from children, it is not a toy. Children may tip over and injure themselves.
- The walker should be handled close to the body, adopting a posture as straight as possible and looking forward.
- The walker should only be used for what has been developed.
- The maximum load of the walker is 135 kg.
- The walker can only be used if it is in perfect condition.
- Make sure before each use that the walker work well. All screws should be tightened.
- The walker seat is only intended for short breaks and is not suitable as a chair.
- Never stand on the walker and do not use it to carry people.
- The surface temperature of the walker must be checked before use. In adverse weather conditions (eg extreme cold, direct sunlight), hot water or visits to the sauna, the surface can reach dangerous temperatures for health.
- Use the walker only on flat, smooth surfaces. Avoid sewers, grates or similar obstacles, as the wheels could get trapped.
- The walker should only be used after being completely checked and in the deployed and locked position.

Make sure that the walker is used only by persons familiar with these instructions.

MOUNTING AND ADJUSTMENTS MOUNTING



OBSERVATION!

Use the walker only once it is fully assembled and in the deployed and locked position! Make sure before use that the product is in perfect condition! Do not use the walker if it has visible damage, or if there is a suspicion that it may be damaged!

1. Remove the packing material
2. Remove the chassis and deposit the walker in front of you, lower the seat.
3. Deploy the structure of the walker and place it on the floor.
4. Mount the tubes of the front and rear legs into the structure by pressing inwards.



Since this model has double independent regulation of height of seat and height of cuffs, regulate the wheels and / or legs in relation to the height to which the seat wishes.



5. Place the handle and set it to the proper height, and secure with the handles.



6. Place the backrest into the structure by pressing on the tubes.



7. Place the basket in the structure. And the assembly is over.

HANDLE ADJUSTMENT



OBSERVATION!

Adjust the handles on both sides to the correct height, using the knobs. Position the walker in the direction of travel and place the arms horizontally. The handles should be at the level of the wrist.

The height of the handle can be adjusted through the four holes. To adjust the height, loosen and remove the screws on both sides and adjust the bar to the desired height. Fasten both screws into the proper hole.

HANDLE ADJUSTMENT



OBSERVATION!

The basket is only intended for the transport of small objects.

Do not transport unpacked items in the bag.

The bag is attached to the walker attach on the seat bar.



SEAT

- The seat is suitable for short breaks but is not suitable as a chair.
- Check before sitting that the walker is well deployed and does not accidentally fold.
- Sit as straight as possible on the seat.
- The backrest adds safety and prevents the walker from overturning.

WALKER'S FOLDING

- Be careful not to pinch Hands folded. Risk of injury !
- Look the seat up and Pull the seat upright.
- So shall the walker of Automatically.



CARE, MAINTENANCE AND STORAGE

○ Repair

- Do not modify the product and do not repair it yourself. Request spares from the manufacturer only. If you repair the walker yourself, unforeseen damage may result.



OBSERVATION !

○ Cautions

- Do not use corrosive or abrasive cleaners or scouring pads. They can affect the surface.
- Remove the walker chassis regularly with a slightly damp cloth. In case of increased dirt, a soapy solution or a mild cleaning agent can be applied. Rinse with clean water. Always dry the walker afterwards, especially the handles.



OBSERVATION !

○ Maintenance

- Before its use it must be verified its operation.
- Wheels: Check the wheels regularly and make sure they do not show signs of wear.
- Table: Check regularly that the frame of the walker has no deformation or scratches.

○ Storage

- Always secure the walker when not in use by folding it
- Do not leave the walker outdoors for a long period of time to avoid damage caused by moisture, dirt or direct sunlight.

Push down lock

OPERATION:

Always engage and disengage wheel locks on both sides simultaneously.

TO SLOW DOWN OR COME TO A TEMPORARY STOP —
Push down slowly by applying weight on the handles until unit slows down or comes to a temporary stop.

TO TEMPORARILY LOCK WHEELS IN POSITION —
Once seated, rear wheels will automatically lock until weight is removed.

TO RELEASE WHEEL LOCKS —
Remove weight until wheels start moving freely.

Lifetime Limited Warranty

Your Mobilita branded product is warrantied to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Mobilita device or this warranty, please contact an authorized Mobilita dealer.



© 2017 Eternity Distribution Pvt. Ltd. All rights reserved

Mobilita is a trademark of
Eternity Distribution Pvt. Ltd.

Chennai, Tamil Nadu, India | Made in China for Mobilita
customercare@mobilita.in | TollFree 1800 121 8485