

Deluxe Padded Raised Toilet Seat

code : M100-PURS



The Mobilita logo, featuring the word "mobilita" in a white, lowercase, sans-serif font. The letter "i" is replaced by a stylized orange figure of a person sitting on a toilet. The "o" has a white dot in the center.

www.mobilita.in
TollFree : 1800 121 8485



www.mobilita.in
TollFree : 1800 121 8485

The Mobilita logo, identical to the one on the left panel, featuring the word "mobilita" in white lowercase letters with a stylized orange toilet seat icon for the letter "i".

PRODUCT DESCRIPTION

Weight Capacity : 190Kg Height : 4" (10cm)

Toilet seat raiser made of polyurethane foam – 100% corrosion proof. Presents no pressure marks after prolonged sitting on the toilet. Tool – free installation – easy to attach to the toilet seat. Smooth surfaces allow easy cleaning. Delivered in practical carrier.

INDICATION

Toilet seat raiser to increase the toilet bowl height. Facilitates sitting down and getting up without assistance. To be used in case of mobility impairment and when use of conventional toilet bowls is impossible.

SAFETY INSTRUCTIONS

Please check the product for possible defects before use. Do not use defective product and contact the manufacturer. Make sure not to exceed the maximum load of 190kg.

PACKAGE CONTENT

- **Carrier**
- **Instructions for use**
- **Toilet seat raiser made of PU foam**

INSTALLATION

Place the toilet seat raiser carefully on the toilet seat. To avoid shifting, the limit stop at the front should be in contact with the inside of the toilet bowl. The toilet seat raiser is simply placed on top of the toilet seat and does not require any screwing on. Please ensure that the toilet seat is not moved during use.

CLEANING

Clean the toilet seat raiser regularly with warm soapy water or mild, non – aggressive cleaning or disinfection agents. Do not use any abrasive / acidic cleaning or scouring agents as they damage the material. We assume no warranty in this case.

Lifetime Limited Warranty

Your Mobilita branded product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Mobilita device or this warranty, please contact an authorized Mobilita dealer.



© 2017 Eternity Distribution Pvt. Ltd. All rights reserved

**Mobilita is a trademark of
Eternity Distribution Pvt. Ltd.**

**Chennai, Tamil Nadu, India | Made in China for Mobilita
customercare@mobilita.in | TollFree 1800 121 8485**