

Raised Toilet Seat

code : M102RS, M104RS, M106RS



The logo for 'mobilita' is written vertically in white lowercase letters. The letter 'o' is replaced by a stylized orange icon of a person sitting on a toilet.

www.mobilita.in
TollFree : 1800 121 8485



www.mobilita.in
TollFree : 1800 121 8485

The logo for 'mobilita' is written vertically in white lowercase letters. The letter 'o' is replaced by a stylized orange icon of a person sitting on a toilet.

Installation Instructions

Weight Capacity : 135Kg

1. Remove product from carton and read directions carefully before installing raised toilet seat.
2. Remove toilet seat from toilet.
3. Turn both locking clamps counter - clockwise until they can no longer turn. This will open the locking devices.
4. Place raised toilet seat onto the bowl and position in a secure fashion . Be sure the two locking brackets are on both sides of the bowl wall and that the seat is flush to the bowl.
5. Turn the locking brackets clockwise until tight. Hand tighten only . Do not use tools as threads on adjustment bracket may strip.
6. Check for secure fit routinely before using product. User's weight should be centered over toilet seat.

Removal instructions

1. Rotate knobs counter - clockwise until seat is loose.
2. Lift front of raised toilet and pull forward to move.

Cleaning Instructions

1. Use a non - abrasive detergent or household cleaner with warm water.
2. Rinse with warm water and wipe dry.
3. **DO NOT USE AN ABRASIVE CLEANER OR CLOTH** with this product, as damage may occur.

CAUTION

1. Always be sure that the seat is correctly and securely locked in place before using.
2. Be sure that adjustment knobs on sides of seat are tight.
3. This product is designed for use with most common types of toilets. It will not fit all toilet types; be sure that the unit is correctly installed, fits properly and is stable before use.

Lifetime Limited Warranty

Your Mobilita branded product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Mobilita device or this warranty, please contact an authorized Mobilita dealer.



© 2017 Eternity Distribution Pvt. Ltd. All rights reserved

Mobilita is a trademark of
Eternity Distribution Pvt. Ltd.

Chennai, Tamil Nadu, India | Made in China for Mobilita
customer@mobilita.in | TollFree 1800 121 8485