

Bedside Commode

Foldable

code : M300



mobilita

www.mobilita.in
TollFree : 1800 121 8485



www.mobilita.in
TollFree : 1800 121 8485

mobilita

Assembling Manual



01. Package Content : 1 commode frame, 1 backrest, 1 seat with lid & 1 bucket with lid.

02. Place commode on the ground and swing legs forward.

03. Swing the bucket holder platform up and insert push buttons on each side into the brackets located on each leg. (Be sure push buttons are fully engaged into place on both sides and check by pulling outward on the legs)

04. Insert the backrest into the brackets located on the rear legs. (Be sure push button is fully engaged on both sides and check by pulling upward)

05. Place bucket into the bucket platform holder.

06. Place the toilet seat onto the frame so that plastic clamps rest on the rear cross frame and on each side of the bucket holder platform. Push down forcefully until brackets are fully attached to the rear cross frame.

07. Be sure toilet seat is in the down position before trying to attach the lid. Place the lid onto the frame so that plastic clamps rest on the rear cross frame and on each side of the toilet seat brackets. Push down forcefully until brackets are fully attached to the rear cross frame.

08. Adjust the height of the legs by locating the push-button on each leg and sliding the leg up or down to the desired height. Make sure all legs are fully engaged and adjusted to equal heights.

Warranty

Your Mobilita branded product is warrantied to be free of defects in materials and workmanship for the product of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Mobilita device or this warranty, please contact an authorized Mobilita dealer.



© 2017 Eternity Distribution Pvt. Ltd. All rights reserved

Mobilita is a trademark of
Eternity Distribution Pvt. Ltd.
Chennai, Tamil Nadu, India | Made in China for Mobilita
customercare@mobilita.in | TollFree 1800 121 8485