

# Drop-Arm Bedside Commode with Padded Arms

code : M301



mobilita


[www.mobilita.in](http://www.mobilita.in)  
TollFree : 1800 121 8485



[www.mobilita.in](http://www.mobilita.in)  
TollFree : 1800 121 8485

mobilita

## Assembling Manual

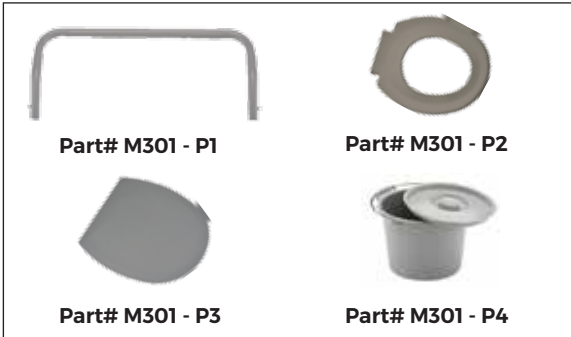

**Package Content :** 2 parts of drop arm commode frame, 2 pcs of connection tubes, 1 backrest, 1 splash guard, 1 seat lid, 1 seat, 1 bucket with lid

**01.** Remove contents from box. Depress brass push pins and insert bucket frame section into first arm. The push pins will snap into place when frame is securely in place.

Note: Make sure the back of the bucket frame (the side with the back rest receptacles) corresponds to the back of the armrest section.



**02.** Repeat for second arm section.



**03.** Insert backrest into backrest receptacles in commode bucket frame. The brass push pins will snap into place when secure.



**04.** Align the padded toilet seat clamps onto the back frame rail. (Both clamps should sit outside the pail brackets). Push down on seat until clamps snap firmly into place. To remove, grasp the back of the seat and lift



**05.** . Install commode bucket by dropping into bucket frame. Reverse to remove.



## Adjusting leg height

**01.** Before use, adjust height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes, per illustration above.

**02.** To be sure legs are locked into place by pushing down on or by pulling legs.

## Additional precautions

**01.** Toilet seat must be in down position before use. Failure to do so may result in breakage of commode and possible injury.

**02.** Do NOT install commode without first reading and understanding this assembly and instruction sheet. If you can not understand these instructions, contact a health-care, dealer or technical professional before attempting to install. Otherwise, injury or damage may occur.

**03.** Inspect all parts for shipping damage before assembly. If there is shipping damage - DO NOT USE. Contact dealer/carrier for further instructions.

**04.** Make certain that the height adjusting snap buttons fully protrude through the same respective hole of each leg extension. This will ensure that the leg extensions are securely locked in position and an even height is achieved (see Figure 1).

**05.** Ensure ALL screws, nuts and/or bolts are tightened.

**06.** Check the rubber tips on the leg extensions for rips, wear or if they are missing. Immediately replace any or

all if any of these imperfections exist.

**07.** All four leg extensions with rubber tips **MUST** touch the floor simultaneously at all times.

**08.** Users with limited physical strength should be supervised or assisted while using commode.

## Warranty

Your Mobilita branded product is warranted to be free of defects in materials and workmanship for the product of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Mobilita device or this warranty, please contact an authorized Mobilita dealer.



© 2017 Eternity Distribution Pvt. Ltd. All rights reserved

Mobilita is a trademark of  
**Eternity Distribution Pvt. Ltd.**

Chennai, Tamil Nadu, India | Made in China for Mobilita  
customer@mobilita.in | TollFree 1800 121 8485