

Deluxe Heavy Duty

Drop-Arm Commode

code : M303



mobilita

www.mobilita.in
TollFree : 1800 121 8485



www.mobilita.in
TollFree : 1800 121 8485

mobilita

Commode assembly instructions and precautions

Weight Capacity : 136 Kg



Bucket & Lid




Splash Guard



Back Rest



Leg Extensions

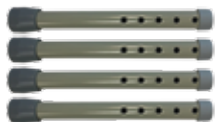

- 01. Remove all parts from box -**
1 drop arm commode with seat and lid,
1 bucket with lid,
1 splash guard,
1 backrest, and 4 leg extensions.

02. To adjust the legs and seat height, place the seat face down and depress the silver push button on one of the legs of the commode and slide a leg extension over it. Lengthen or shorten the leg extension to the desired position. Once the desired position is found, ensure that the silver push button protrudes from the adjustment hole of the leg extension and is fully engaged so that it does not move. Adjust the other three legs to the same length, and make sure that the commode sits level.

03. Lay the front end of the commode face down and Insert the backrest into the tube slots under the seat. Depress the silver push buttons on each side of the backrest until they protrude from the adjustment hole and are fully engaged into the user's desired setting. Ensure the silver push buttons are fully engaged by pulling on the back.

04. Lift the seat and place the bucket in the center of the commode in between the bucket hangers. When using the commode over a toilet, replace the bucket with the splash guard.

HOW TO USE YOUR Drop Arm Commode

01. Lowering the drop arms: On the side of the arms, push in the RED spring loaded release button and push down on the drop arm.

Raising the drop arms: Pull up on the arms until the RED spring loaded release button completely pops out and is locked.

02. The commode should be low enough to ensure that your feet touch the floor when sitting on it, but high enough to enable easier sitting and standing.

03. After use, remove the bucket, discard waste and sanitize.

Warranty

Your Mobilita branded product is warranted to be free of defects in materials and workmanship for the product of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Mobilita device or this warranty, please contact an authorized Mobilita dealer.



© 2017 Eternity Distribution Pvt. Ltd. All rights reserved

Mobilita is a trademark of
Eternity Distribution Pvt. Ltd.

Chennai, Tamil Nadu, India | Made in China for Mobilita
customercare@mobilita.in | TollFree 1800 121 8485