Foldeasy Toilet SurroundSafety Frame

code : M304



www.mobilita.in TollFree : 1800 121 8485

Product Description

Weight Capacity: 100 Kg



his toilet aid frame is free assembling, supporting aid when using commode, with CE approval

4.2 kg
100 kg
18"
20"

Height 27.50" - 35.40"

Instruction

Before use, please fully understand the product and applicability.

01. Take out the product, unfold to normal size



O2. Height adjustable. Put the bolt out and extend the tube to suitable height



 $\textbf{Attention:} \ \mathsf{Keep} \ \mathsf{four} \ \mathsf{legs} \ \mathsf{height} \ \mathsf{to} \ \mathsf{equal} \ \mathsf{standard}.$

03. Put the board down to the safety pins



04. After use, put the board up then press the buttons to fold the frame



Cleaning and Maintenance

When, cleaning, use warm water or neuter cleaner. Do not user abrasive paper and rough materials

Warranty

Your Mobilita branded product carries a limited warranty to be free from defects in material and workmanship. Applicable to the original purchaser only.

This device is built to exacting standards and carefully inspected prior to shipment. This Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Mobilita device or this warranty, please contact an authorized Mobilita dealer.



© 2017 Eternity Distribution Pvt. Ltd. All rights reserved

Mobilita is a trademark of Eternity Distribution Pvt. Ltd.

Chennai, Tamil Nadu, India | Made in China for Mobilita customercare@mobilita.in | TollFree 1800 121 8485





CE

www.mobilita.in TollFree : 1800 121 8485