

6 in 1 Multi Funcional

Walker / Commode / Shower chair

code : M305

www.mobilita.in
TollFree : 1800 121 8485

The logo for 'mobilita' is oriented vertically. It features the word 'mobilita' in a white, lowercase, sans-serif font. The letter 'o' is replaced by a stylized orange icon of a person sitting in a wheelchair, with a white wheel. The letter 'i' has a small orange dot above it. The letter 'a' has a small orange dot above it.

Product Description

Weight Capacity : 100 Kg



Overall Height : 30 - 37.8"

Overall Width : 21"

Overall Depth : 18"

Overall Product Weight : 6.4 Kg

Instructions and Illustrations

01. As a commode chair, the instructions and installation Of M305



- Open the buckle - Put seat button on the support tube, forced to press the seat plate into that.
- Hard to press the next buckle cover, can be fastening. - Move the seat plate to the middle of the support tube.
- Put the bucket into buckle, ensure the installation is secure, then can be used.

02. As a Shower chair, the instructions and installation Of M305



Remove the bucket, put a small cover, then the shower chair can be used.

03. As a walker, the instructions and installation Of M305

Remove the bucket and sitting plate, you can press the bullet sleeve, and turn the armrest inward, the walker immediately convertible.



When close, press down.

Rotate the handrail.

When the release button are on the upper/ lower slot, the walker is fixed/ moved.

Press the release button to adjust the height of foot tube.

04. As a handrail, the instructions and installation Of M305



The walker on the sofa, bedside, the edge of the toilet, etc., the user will be able to use the short handrails to easily stand up.



05. As a toilet seat raiser, the instructions and installation Of M305



Adjust the height of the foot tube to achieve the toilet seat raiser function.

After the increase, please sit gently, to avoid shaking

Use-Range

The assisting frame acts as a support for those who are inconvenient in walking, and when the force is applied to the assisting frame, the walker will move slowly forward to complete the function of helping to walk.

Attention :

- Products should be smooth, not strong shaking, to prevent damage to the structure.
- Regularly check the situation at the junction of the assisting frame to prevent the movement of the marbles from falling off.
- Wheelchair rated load of 100kg, the user weight more than 100kg Please be careful to use or contact manufacturers for special custom.
- Products from the date of the factory, the normal life of 3 years.

Product Maintenance

01. Do not scratch the product surface to prevent damage to the material.
02. Daily cleaning and maintenance, the application of neutral synthetic detergent, wipe with a damp cloth, and then wipe clean with a dry cloth.
03. Daily routine cleaning and disinfection once one day.
04. Do not contact with strong acids, strong alkali, to prevent corrosion.
05. Suggestions that when it is used as a shower chair, it should be wiped with the dry cloth.

Special Storage Conditions

After the package, the walker should be stored in the temperature which does not exceed $-10 \sim 40$, and relative humidity does not more than 80%. Non-corrosive gas and well ventilated, clean interior also

Transport should be moisture, to avoid light, away from heat.

Warranty

Your Mobilita branded product carries a limited warranty to be free from defects in material and workmanship. Applicable to the original purchaser only.

This device is built to exacting standards and carefully inspected prior to shipment. This Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Mobilita device or this warranty, please contact an authorized Mobilita dealer.



© 2017 Eternity Distribution Pvt. Ltd. All rights reserved

Mobilita is a trademark of
Eternity Distribution Pvt. Ltd.

Chennai, Tamil Nadu, India | Made in China for Mobilita
customer@mobilita.in | TollFree 1800 121 8485



www.mobilita.in
TollFree : 1800 121 8485

The logo for 'mobilita' features a stylized orange figure of a person riding a bicycle, positioned above the word 'mobilita' which is written in a white, lowercase, sans-serif font.