

# Foldable Aluminum Shower Chair

code : M400



The logo for 'mobilita' is oriented vertically. It features the word 'mobilita' in a white, lowercase, sans-serif font. The letter 'o' is replaced by a stylized orange icon of a person sitting in a chair, with a white circle representing the seat and a white dot for the person's head.

[www.mobilita.in](http://www.mobilita.in)  
TollFree : 1800 121 8485



[www.mobilita.in](http://www.mobilita.in)  
TollFree : 1800 121 8485

The logo for 'mobilita' is oriented vertically. It features the word 'mobilita' in a white, lowercase, sans-serif font. The letter 'o' is replaced by a stylized orange icon of a person sitting in a chair, with a white circle representing the seat and a white dot for the person's head.


# Foldable Aluminum Shower Chair

**This instructions is to be perused carefully and must be always accessible to all users of the product.**

## Indication

Standing insecurity / standing inability with decrease or missing functional ability of the lower extremities or other illnesses which are affecting the stability (e.g., strength loss, chronic, advanced heart circulatory illnesses and lung illnesses) infers with a body position weakness, so that an independent use of the shower is still allowed.

## Parts of Shower chair

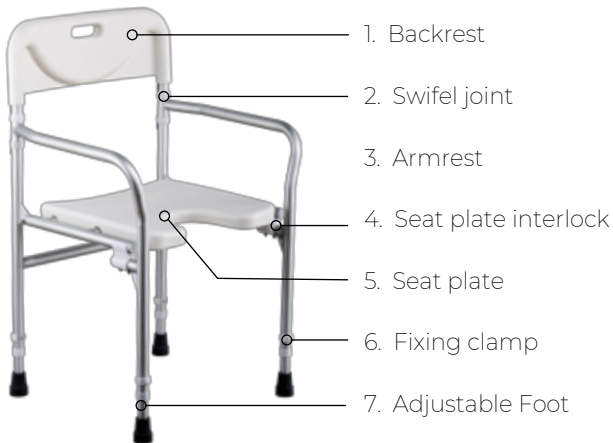

- Swing both side parts forwards and then the seat record down them on the hold clips lies .
- Press now the seat plate hard down until it engages audible in the hold clips on the left and on the right side .
- Check the correct position of the seat plate in the hold clips.
- Height adjustment of footrests
- To adjust the footrests in the height first unlock the fixing clamp. Then the footrests can be shifted in 4 steps in the height for seat heights from 45.7-53.3 cm / 18" - 21" inches

## Cleaning / Maintenance

- Clean regularly by hand with a damp cloth and usual household cleaning agents !
- Do not use any aggressive detergents !
- Screws of must be checked regularly and if necessary retightened.

## Reapplication

This product is suitable for reapplication. With the processing the standards of the manufacturer must be considered. These standards are made available on inquiry.

## Safety instructions

- Read this instructions carefully.
- Before use all components are to be checked for damages and firm seat.
- While seating, rising and while folding up or engaging the seat plate it` s possible to pinch clothing or body parts
- - The steadiness of the shower chair is clearly lower as with a normal chairs, therefore avoid lateral lean out – Tilting danger !
  - The max. user weight 113 kg on the seat!

## Assembly

The shower chair is completely assembled. Unfold him as illustrated and put the seat upwards.



## Warranty

Your Mobilita branded product is warranted to be free of defects in materials and workmanship for the product of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Mobilita device or this warranty, please contact an authorized Mobilita dealer.



© 2017 Eternity Distribution Pvt. Ltd. All rights reserved

Mobilita is a trademark of  
**Eternity Distribution Pvt. Ltd.**

Chennai, Tamil Nadu, India | Made in China for Mobilita  
customercare@mobilita.in | TollFree 1800 121 8485