

# Deluxe Aluminium Shower Commode Chair

code : M401



The logo for 'mobilita' is written vertically in white lowercase letters. The letter 'o' is replaced by a stylized orange icon of a person sitting in a chair, with a white circle representing a wheel.

[www.mobilita.in](http://www.mobilita.in)  
TollFree : 1800 121 8485



[www.mobilita.in](http://www.mobilita.in)  
TollFree : 1800 121 8485

The logo for 'mobilita' is written vertically in white lowercase letters. The letter 'o' is replaced by a stylized orange icon of a person sitting in a chair, with a white circle representing a wheel.

# Deluxe Aluminum Shower Commode Chair



## INSTRUCTIONS

### 1. Introduction

Featuring soft feel polyurethane moulded seat and upholstery for comfort, warmth and durability. Clip-on building integrated seat easy for daily cleaning. Flip-up armrest for easy transfer. Multi-use function as a shower chair, commode or over a toilet pedestal. 4 aluminium 5" castors with brake. Swing away footrests.

### Specifications

Unit Weight	11 Kg
Weight Capacity	136 Kg
Width of handgrips	18"
Seat Area	18" – 17"
Seat Height	18.3" x 21.3"
Overall Dimensions	37.8" (L) x 21" (W)
Overall Dimensions Height	35.4" - 38.3"
Color	Powder coated white

### 2. Contents of the Box

- Attendant Shower Commode Chair (T210)
- Footplates (T210FOOT)
- User Instructions (this leaflet)

### 3. Assembly Instructions

- Carefully unpack and examine the product to ensure it is complete.
- Place the assembled base on a level floor. At the rear of the commode you will see two black threaded wheel nuts. Undo the wheel nuts so that the locking holes are clear.
- Place the two legs of the back into the locating holes on the back edge of the commode frame.
- Secure the back by turning the black lock nuts until you feel the nut biting against the leg of the back. **DO NOT OVERTIGHTEN -FINGER TIGHT IS SUFFICIENT.**

- Place the two detachable arms into the four locating holes – two each side of the commode.

- Before use check for any signs of damage e.g. cracked weld, flaking of paint, bent tube. If any signs occur please contact your supplier to exchange and refrain from using the product. **DO NOT ATTEMPT TO USE IF PRODUCT IS DAMAGED.**

#### **4. Cleaning**

- Use mild detergent with warm water to wipe down framework on at least a weekly basis more frequently if required.

- Do not use an abrasive cleaner as this will damage the frame and nullify your warranty.

- Ensure that all detergents are washed off thoroughly.

- Dry the product, paying particular attention to all joints and abutments and that the frame contains no trapped water.

#### **Use of power/jet washing & industrial washing machines:**

- If either of the above is used to clean equipment care must be taken since high-pressure water can be penetrative.

- Care should be taken that frames are not placed too close to high-pressure water jets.

- After cleaning ensure any residual water is removed from the framework as it could cause permanent rusting of the frame (leave frames open in a warm environment for several hours to ensure all internal

moisture evaporates).

- We cannot offer advice on industrial washing machines with regards to specific machine settings, as these will be dependant on the manufacturer's instructions.

#### **5. Maintenance Requirements**

- The product should be checked on a regular basis for damage or wear and tear – this includes flaking or cracking paint. Any undue signs of damage should be noted and reported to the supplier in the first instance. Should damage be evident, then the user should refrain from using the product until advice has been sought from the supplier.

- It is recommended that all joints, abutments and castors are sprayed with a water dispersing agent such as WD40. This will give your commode greater longevity and avoid premature rust.

#### **6. Warning**

- This product is intended for users of a maximum of 136kg in weight.

- We recommend a full risk assessment should be carried out before use.

#### **7. Disposal Instructions**

- Appropriate hygiene standards should be applied before disposal to reduce the risk of infection.

- Materials used are essentially non hazardous and can be disposed of appropriately.

## Warranty

Your Mobilita branded product is warranted to be free of defects in materials and workmanship for the product of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Mobilita device or this warranty, please contact an authorized Mobilita dealer.



© 2017 Eternity Distribution Pvt. Ltd. All rights reserved

Mobilita is a trademark of  
**Eternity Distribution Pvt. Ltd.**  
Chennai, Tamil Nadu, India | Made in China for Mobilita  
customercare@mobilita.in | TollFree 1800 121 8485