

Luna Shower Chair

with arm rest

CODE M406



www.mobilita.in
TollFree : 1800 121 8485

Product Description

Weight Capacity : 227 Kg




Mobilita Shower Chair is a product of quality engineering and is manufactured to meet rigid specifications.

Please read this manual carefully to ensure safe usage and understanding of maintenance instructions.


Assembly Instruction

Check all parts

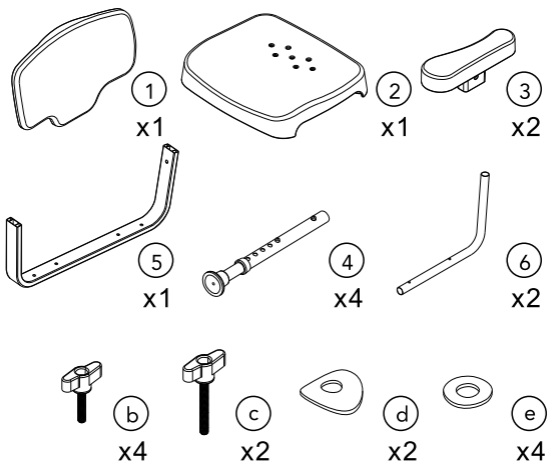

Insert each leg ④ into the seat.
(Caution: For secure installation, make sure the button on each leg is fully engaged to the hub on the seat.)


Step 1

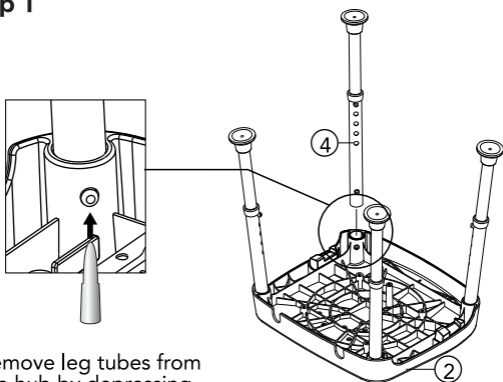

Remove leg tubes from the hub by depressing push button.


Fix the two points of tubes ⑥ onto seat with parts ①(b) & ①(d) .

Step 2

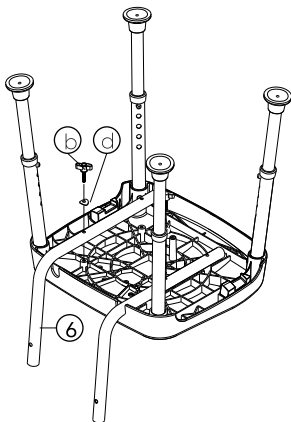


Fix the middle point of hand rest tube onto back rest tube with parts ①(c) & ①(e) . Fix the outside point of hand rest tube with parts ①(b) & ①(e) .

Step 3

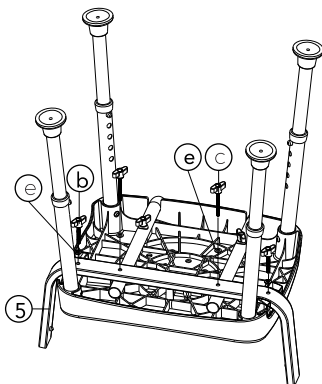


Install the backrest onto back tubes.

Step 4

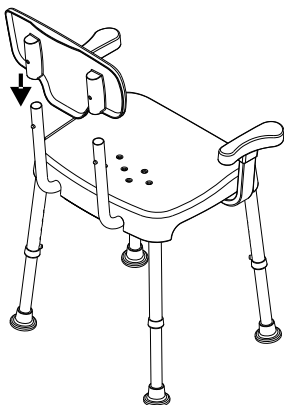

Install the hand rest onto hand rest tube. (Caution: For secure installation, make sure the button on each hand rest tube is fully engaged to the hub on the hand rest.)

Step 5

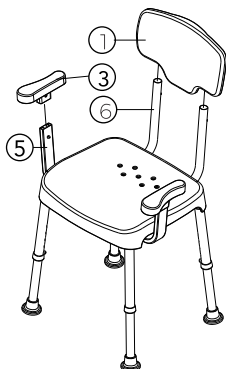

Caution

- ⦿ Read instruction before assembly.
- ⦿ Please check before assembling if the parts quantities are correct.
- ⦿ Should be checked before each use to make sure it is securely assembled. Reinstall if necessary.
- ⦿ Max User Weight 227 kgs
- ⦿ Do not sit alone on the corner of the seat to avoid injuries.
- ⦿ Mobilita assumes no liability for any damage or injury caused by improper assembly or use of this product.

Warranty

Your Mobilita branded product carries a limited warranty to be free from defects in material and workmanship. Applicable to the original purchaser only.

This device is built to exacting standards and carefully inspected prior to shipment. This Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Mobilita device or this warranty, please contact an authorized Mobilita dealer.



© 2018 Eternity Distribution Pvt. Ltd. All rights reserved

Mobilita is a trademark of
Eternity Distribution Pvt. Ltd.

Chennai, Tamil Nadu, India | Made in Taiwan for Mobilita
customer@mobilita.in | TollFree 1800 121 8485



www.mobilita.in
TollFree : 1800 121 8485

mobilita®
The logo features a stylized orange figure of a person riding a bicycle, positioned behind the lowercase word 'mobilita' in a white, sans-serif font. A registered trademark symbol (®) is located at the top right of the word.