

Seat Lift Assist

restore confidence !

code : M501

www.mobilita.in
TollFree : 1800 121 8485



Seat Lift Assist

Weight Capacity : 100 Kg



SAFETY PRECAUTIONS

The unique design of the Mobilità Seat Lift Assist has safety at its core. While Mobilità is made to be as safe as possible, there are some precautions you need to take:

01. Make sure you have something to hold on to when getting up. Mobilità Seat Lift Assist is designed for use on chairs with arms that offer a suitably stable seating surface. Mobilità is not recommended for use in wheelchairs, automobiles, reclining chairs or office chairs with non-lockable wheels.
02. Always make sure that the chair your Mobilità Seat Lift Assist is used on will not slide along the floor. When in doubt, place the chair with its back against a wall or other fixed object.

03. When bending over to pick something up, do not put any weight on your feet – otherwise the Mobilità will start to lift you.

PLEASE NOTE : If, after setting your Mobilità to the proper weight setting, the cushion does not go down when you sit on it, the lifting mechanism may be stiff from prolonged inactivity. In this case, simply set the weight setting at the lowest level, and sit on the cushion. A few times until you feel it moving freely again. Then reset for your weight.

PART IDENTIFICATION



SELECT WEIGHT SETTINGS

01. Stand Mobilità on the end opposite the handle.
02. Push down on the upholstered part of the cushion, causing it to flex outward. This allows the lifting mechanism to swing freely.

(The Mobilità Seat Lift Assist cushion is designed to flex this way thousands of times. Flexing will not damage the product.)

03. Select a weight setting. Center the lifting mechanism pin between the two grooves for user's appropriate weight. Make sure it is fitted securely in the center of the weight slots.

(The printed numbers on the setting gauge are only a guideline. The setting you choose should be according to the user's ability and preference. Choosing a weaker setting will still allow a gentle lowering into the chair but will not provide as much force on rising.)

CLOSING FOR TRANSPORT

01. Using the method for adjusting the weight settings, allow the lifting mechanism to swing below the bottom setting.
02. Close the Mobilità Seat Lift Assist cushion and wrap the Velcro strip around the Velcro pad located on the carrying handle.

PLACEMENT IN THE CHAIR

Generally, the Mobilità Seat Lift Assist should overhang the front of the chair as shown. On chairs which are very deep, it is permissible to position the cushion further back. When using the Mobilità Seat Lift Assist, keep both hands on the arms of the chair (or other suitable support).

SITTING DOWN WITH MOBILITÀ SEAT LIFT ASSIST

After adjusting the weight settings and placing the Mobilità Seat Lift Assist on your chair, sit down on the flat part at the top of the cushion. The Mobilità Seat Lift Assist will ease you slowly and gently into your chair.

Note: If the cushion does not go down when you sit on it, re-adjust the weight setting to a lower weight.

GETTING UP WITH MOBILITÀ SEAT LIFT ASSIST

When you are ready to stand, simply start to get up like you normally would, by leaning slightly forward and pushing off with your arms and legs. Mobilità Seat Lift Assist will sense you are getting up and give you a slow and gentle lift up.



www.mobilita.in
TollFree : 1800 121 8485

The logo for Mobilita, featuring a stylized orange figure of a person riding a bicycle, positioned above the word "mobilita" in a white, lowercase, sans-serif font.

CARING FOR YOUR SEAT ASSIST

The cover on your Mobilitàà Seat Lift Assist is made of 100% polyester and can be removed and machine washed (hang to dry).

The surface of the Mobilitàà Seat Lift Assist cushion is coated with a special water-resistant coating. This may be sponge cleaned with mild soap or detergent.

Note : Never totally immerse your Mobilitàà Seat Lift Assist in water.

To put on the cover after cleaning:

01. Adjust the weight setting on your Mobilitàà Seat Lift Assist and place it on a table with the open end facing away from you.
02. Place the end of the cover with the tag over the back or top of the cushion and pull it down and under the front edge.

Note : The back or top end of the cover is made wider than the front to ensure proper fit. It is best to leave all or part of the tag on your cover so that it is easy to identify this back/top end in the future.

Warranty

Your Mobilita branded product carries a limited warranty to be free from defects in material and workmanship. Applicable to the original purchaser only.

This device is built to exacting standards and carefully inspected prior to shipment. This Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Mobilita device or this warranty, please contact an authorized Mobilita dealer.



© 2017 Eternity Distribution Pvt. Ltd. All rights reserved

Mobilita is a trademark of
Eternity Distribution Pvt. Ltd.

Chennai, Tamil Nadu, India | Made in China for Mobilita
customer@mobilita.in | TollFree 1800 121 8485