

# Adjustable Back Rest

For your comfort

code : M506



[www.mobilita.in](http://www.mobilita.in)

TollFree : 1800 121 8485

## Adjustable Back Rest

Weight Capacity : 113 Kg



## Introduction

**Congratulations !** You have purchased Adjustable Back Rest from Mobilita which is manufactured from the finest material and components available, when installed and used correctly it is designed to give many years of reliable trouble-free service.

The Adjustable Back Rest can be used on single, double, queen and king size beds

## Before Use

Carefully remove packaging. Avoid using knives or other sharp instruments as this may damage the surface of the product.

**01**

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

**Please Note :** Part of the product packaging is plastic, please take care when handling and keep the packaging well away from children and babies to avoid the danger of suffocation.

Please dispose of all packaging carefully.

## Assembly and Instruction

1. Place the Mobilita back rest onto the bed squarely with the blue fabric covered support section facing upwards and the small padded head rest at the top of the bed.
2. Raise the blue fabric covered support section of the back rest to the required height.
3. Insert the hinged holding bar on the back of the support section into one of the sets of plastic hook stays on the base of the product.
4. Ensure that the bar is seated fully on both of the hook stays before use.

## Usage and Maintenance Instructions

**HEIGHT ADJUSTMENT :** The Mobilita Adjustable Back Rest is designed to give you many years of service, however please note important information below to ensure your product remains in good working order:

- Do not exceed the weight limit of 113kg as this will compromise the safety of the user

- Do not share the backrest with anyone; it is designed for one person only.
- Do not place obstacles or limbs between the opening section of the back rest at any time.
- Check that the back rest is properly locked in the plastic hook stays when in use.
- Keep naked flames, cigars and cigarettes well away from the back rest.

**CLEANING** : It is important that you only clean the product using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners. Abrasive cleaning pads will seriously damage the product beyond repair. Always ensure that you wipe dry the equipment after cleaning.

## Important Information

The information given in this instruction booklet must not be taken as a part of or establishing any contractual or other commitment by Mobilita or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly / use of your product.

## Warranty

Your Mobilita branded product is warranted to be free of defects in materials and workmanship for the product of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Mobilita device or this warranty, please contact an authorized Mobilita dealer.



© 2017 Eternity Distribution Pvt. Ltd. All rights reserved

Mobilita is a trademark of  
**Eternity Distribution Pvt. Ltd.**

Chennai, Tamil Nadu, India | Made in China for Mobilita  
customer@mobilita.in | TollFree 1800 121 8485



[www.mobilita.in](http://www.mobilita.in)  
TollFree : 1800 121 8485

The logo for Mobilita, featuring a stylized orange figure of a person riding a bicycle, positioned above the word "mobilita" in a white, lowercase, sans-serif font.